


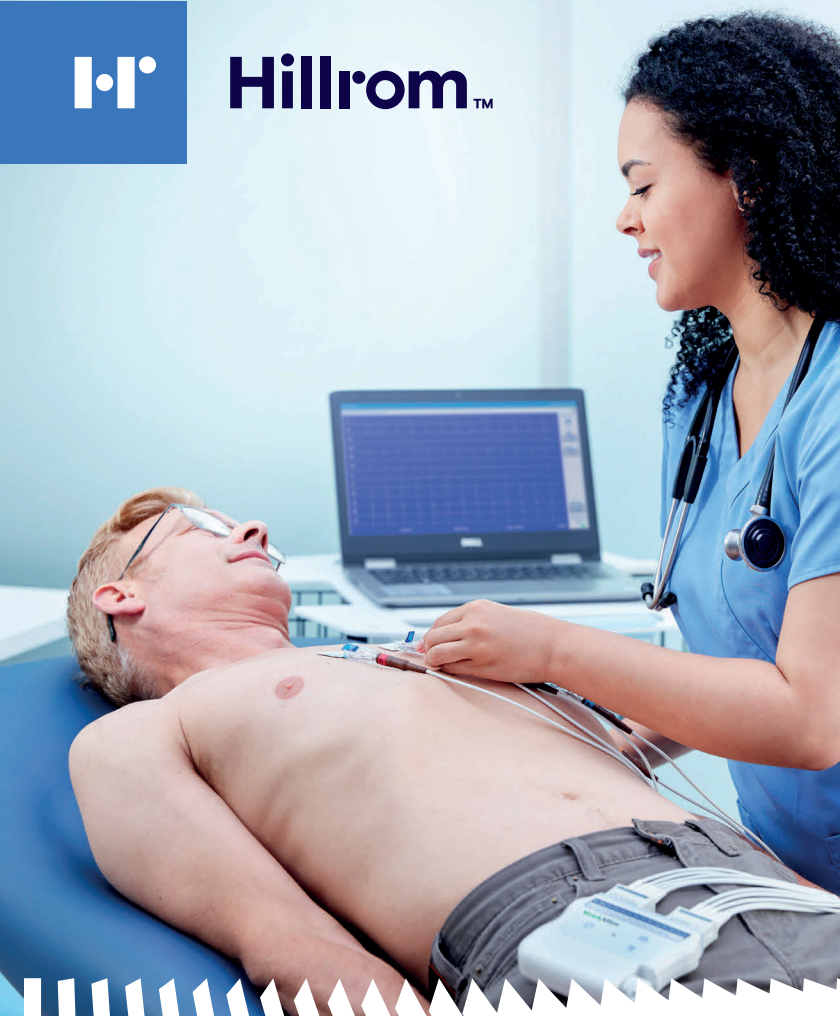

Hillrom™



ADVANCING CONNECTED CARE™ WITH ECG CONNECTIVITY

Hillrom and athenahealth®

We're helping you improve clinical workflows and reduce manual documentation for ECG data by sending it right to your EMR. See how the Welch Allyn® Diagnostic Cardiology Suite™ ECG can help create a workflow that works best for your practice.

Unconnected Workflow

- 1 Order an ECG
- 2 Manually document patient demographics in the ECG software
- 3 Capture ECG and print test results for physical review
- 4 Add physician notes to ECG printout
- 5 Scan the ECG printout and import into athenahealth
- 6 Add diagnosis in athenahealth
- 7 Complete order

Connected Workflow

- 1 Order ECG in athenahealth
- 2 Launch Diagnostic Cardiology Suite ECG from athenahealth (patient demographics will automatically populate and acquisition will start)
- 3 Capture ECG; image of unconfirmed ECG is uploaded to athenahealth
- 4 Physician opens ECG from athenahealth by clicking "Update Results" to review, then edits the interpretation and signs
- 5 Final report (and discrete data) uploaded to athenahealth

Connectivity Counts

Help save time and improve data accuracy by streamlining your clinical workflows.

- Reduce time previously spent manually entering data into the EMR
- Improve accuracy by automatically associating results with the EMR orders
- Work more efficiently by freeing up your time for other patient priorities



A clinician launches the Diagnostic Cardiology Suite ECG from athenahealth®.



An ECG is captured* and saved for physician review, edit and signature.



The final report with discrete data is uploaded to athenahealth.

Contact your Hillrom representative to learn more.



hillrom.com

*A Wireless Acquisition Module or an AM™ 12 Module (via a USB connection) can be used with the Diagnostic Cardiology Suite to capture an ECG.

Hillrom reserves the right to make changes without notice in design, specifications and models. The only warranty Hillrom makes is the express written warranty extended on the sale or rental of its products.

© 2020 Welch Allyn. ALL RIGHTS RESERVED. APR108001 Rev 1 80023932 Ver A 06-AUG-2020