

Comprehensive Bariatric Care

Barton Transfer Chair



Hospitals are adjusting to accommodate an increasing population of bariatric patients. To help minimize the risks associated with manual handling of obese individuals, staff must be aware of what equipment is available, how to access it quickly, and trained to use with bariatric patients. Unfortunately, many healthcare facilities lack equipment and protocols for lifting and moving bariatric patients.

Hill-Rom's comprehensive solutions help safely streamline routine tasks and provide a comfortable experience for both patients and caregivers.

Key features here

- The Barton Transfer Chair easily converts between a flat and upright mobile chair that allows caregivers to easily transfer patients.
- Mobile chair featuring unlimited number of reclined and tilt positions.
- Tilt-in-Space capability
- Trendelenburg position capability
- Customized features allow the chair to be set to the patient: adjustable head, leg and arm rests, side supports and height.
- Heavy-duty central locking and steering casters allow for easy mobility.

Comprehensive Bariatric Care

Barton Transfer Chair

How it works—Positioning & Transfer System (PTS™)

- Place the chair at bedside in the horizontal position.
- Attach the sheet to the Positioning & Transfer System (PTS™).
- Transfer the patient by turning a handle.
- Incline the chair and the patient is ready to be transported.
- Reverse the process to easily transfer the patient from chair to bed.



Quickly and easily adjusts to Tilt-in-Space positions.



Fully-adjustable headrest with side support.



Heavy-duty central locking/steering casters.



The optional Barton PTS™ helps transfer your patient with ease.



Manual height-adjustability.

Technical Specifications

Seat Width

I-400*	21" (54.6 cm)
I-660*	27" (68.8 cm)
I-700*	25" (63.5 cm)

Seat Depth

I-400*	19" (48.26 cm)
I-660*	21" (53.3 cm)
I-700*	19" (48.26 cm)

Overall Length

74" (188 cm)**

Back angle 0-70°

Tilt angle 0-17°

Caster Diameter 6"

Fire Safety CAL 117

Maximum weight capacity

I-400*	400 lbs (181 kgs)
I-660*	660 lbs (300 kgs)
I-700*	700 lbs (318 kgs)

* I-400 (capital only); I-700 (capital only); I-660 (rental only)

** Fully extended with headrest and footrest not extended

Hill-Rom reserves the right to make changes without notice in design, specifications and models. The only warranty Hill-Rom makes is the express written warranty extended on the sale or rental of its products.

©2015 Hill-Rom Services, Inc. ALL RIGHTS RESERVED.
155817 rev 3 10-NOV-2015 ENG - US

Barton is a registered trademark of Barton Medical Corp.

For further information about this product or a service, please contact your local Hill-Rom representative or visit our webpage:

USA 800-445-3730
Canada 800-267-2337

www.hill-rom.com

Enhancing outcomes for patients and their caregivers:

Hill-Rom